



Instructions for Use (IFU)

Suction Connecting Tube

Product Description

The Suction Connecting Tube is a sterile, single-use medical device designed for fluid suction and evacuation during surgical, emergency, and ward procedures. It features smooth or ribbed walls, universal connectors, and is available in various lengths and diameters.

Intended Use

Used to connect suction sources to suction tips or canisters for removing fluids, mucus, or blood during clinical or surgical procedures.

Instructions for Use

1. Verify the package integrity before use.
2. Remove the tube from its package in an aseptic area.
3. Connect one end to the suction handle or tip and the other to the suction source or collection canister.
4. Adjust suction pressure as required.
5. Dispose of the tube safely after use.

Cautions

- Do not reuse; single-use only.
- Avoid kinking or bending during use.
- Check that connections are secure before suctioning.

Warnings

- Do not use if the package is damaged or expired.
- Improper use may lead to patient injury or contamination.
- Keep away from children.

Sterilization

Sterilized by Ethylene Oxide (EO). Do not re-sterilize.

Storage Conditions

Store in a cool, dry place, away from direct sunlight and sources of heat. Keep in its original packaging until use.

Contact Information

DIYI Medical
Tel: +8618237371319

Email: info@diyimedical.cn